- GB** Shower bench - user manual
- D** Duschhocker - Gebrauchsanweisung
- DK** Badebænk - Brugervejledning
- I** Sgabello da doccia - Manuale d'uso

Item no.
Art. Nr. **302050**
Varenr.
Codice

User manual

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

GB Shower bench 302050

D Duschkocker 302050

Description & intended use

The shower bench (img. A) is used for seating aid in the shower. The bench has an aluminium frame with height-adjustable legs and a seat made of white plastic.

Important safety information

Only use the device for the intended purpose. Do not climb on the device. Make sure that all legs have the same height. Use only in good technical condition and on a flat and smooth surface. Always check the stability of the device before use.

Assembly and usage description

The device must be assembled by mounting the two crossing legs to the downside of the bench. Mount the leg first that has a flat side on the middle on the downside. Then make sure the last cross-leg fits on the flat side (img. B) and fasten all screws. Adjust the height to fit the user by pulling on the legs and pressing the pin-buttons on each leg to the desired height. Make sure the pins click in place on each leg to lock the position and that all legs are adjusted to the same height (img. C).

Cleaning & maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents and soft cloth. Do not use solvents or abrasives.

Technical data	
Dimension (W x D)	50 x 30 cm
Seat height adjustable	38 - 50 cm
Max. user weight	115 kg
Product weight	1,79 kg

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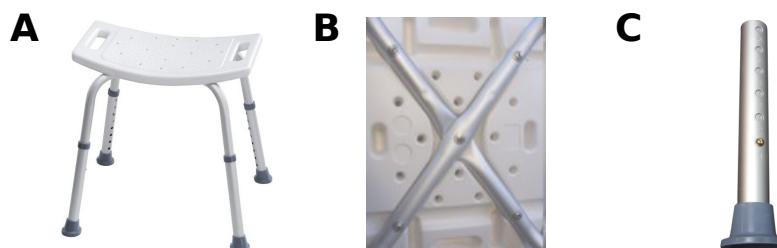
is designed and made in accordance with the technical documentation which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment**

Warranty terms

Mobilex A/S offers 24 months of warranty for damages caused by errors in production. The warranty is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing error. In the event of damage, please notify your dealer. The warranty does not cover transport costs, injury or other damages possibly related to a malfunction of the product.

Lars Nygaard
Managing Director
Mobilex A/S

Skanderborg, 22-03-2018

.....
signature

 MOBILEX A/S Grønlandsvej 5 DK - 8660 Skanderborg Tel: +45 87 93 22 20 www.mobilex.dk	Shower bench  ↓ Max. 115 kg
 Produced <input type="text"/>	Serial no. <input type="text"/>

Example of the product label

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The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark, declares that the: