- GB Pedal trainer**
- D Pedaltrainer**
- DK Pedaltræner**
- I Pedaliera**

Item no.
Art. Nr.
Varenr.
Codice

272002 & 272003

User manual

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

GB Pedal trainer - item no. 272002

Product description

Foldable pedal trainer with a 2-colored steel frame. The pedals have safety straps and the pedal resistance can be adjusted.

Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer don't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose and only on even and flat surfaces.
- Do not stand on the pedal trainer.
- Use only sitting in a chair with the pedal trainer placed safely on the floor or by training of the arms, only placed safely on a surface of the right height.

Assembly

- Remove the device from the box and insert the adjustment knob.
- Turn the legs outwards until they both click in place.

Adjusting the pedalling resistance

The adjustment knob is for adjusting the resistance in the pedals. Turn the knob clockwise. Tighten or loosen as desired.

WARNING!

Device may get hot by extended use!

Cleaning & Maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives).

Guarantee terms

Mobilex A/S offers 2 years of guarantee for failures caused by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

Technical data

Pedal trainer	Item no. 272002 & 03
Size (unfolded)	44 x 39,5 x 23 cm
Size (folded)	44 x 27 x 14 cm
Weight	2,16 kg (2,45 kg)
Max. load	100 kg

GB Pedal trainer - item no. 272003

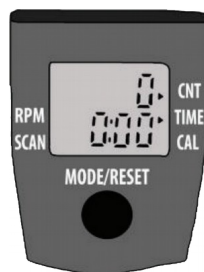
Model with LCD Display

The 272003 model differs from the 272002 model on the additional multifunctional LCD display.

The display provides the following functions:

- 1. Mode** (red button). This control allows you to select a particular function. To start, press and hold the Red Button for 4 seconds. The LCD will display the different functions (after a few minutes without movement, the unit will automatically shut off).
- 2.** To select a function, wait until the 4 second setup is complete. Push the red button and the arrow will move through the functions. When the desired function is selected, simply use the pedal trainer and the unit will calculate

the exercise.

- 3. Time:** Determines the amount of the time the pedal trainer has been operating.
- 4. Counter (CNT):** Determines the amount of steps while operating the pedal trainer.
- 5. Calorie (CAL):** Determines the amount of calories burned while using the pedal trainer.
- 6. Reps per minute (RPM):** Automatically calculates the repetitions per minute.
- 7. Scan:** Will automatically show the results of all functions.

Battery replacement

If the display does not come on or reads incorrectly, the battery requires replacement.

To replace the battery, use a small screw driver and remove the display panel. Turn the display panel over and remove the watch-style battery. Insert a new battery and insert the display panel again.

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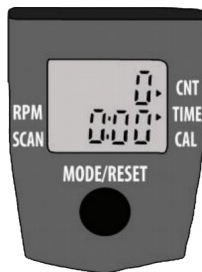
Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

Declaration of Conformity (CE)

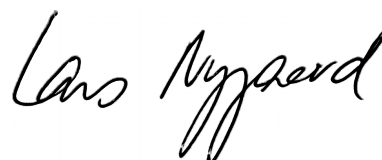
The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Danmark, declares that the

are designed in accordance with the technical documentation and made of which conforms to the following basic requirements:


- **EU Council Directive 93/42/EEC for medical equipment**

Lars Nygaard
Managing Director
Mobilex A/S

.....
signature

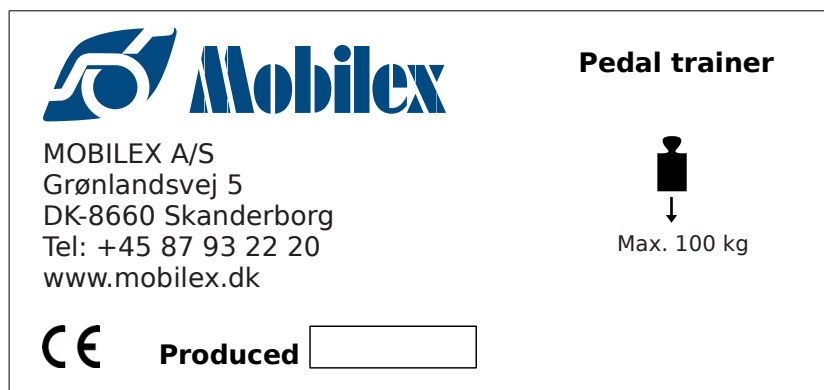Skanderborg, 1/2-2017

Example of the product label