Mobilex wheelchair "Dolphin" & "Barracuda"

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2. Mounting of spoked wheels

Rear wheels are fitted using the supplied wheel axles, axles are released by and pressing the black push button at the bottom.

Note: The balls on the axle must be released on the inside of the bushing otherwise you risk the wheel falling off while driving.

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3. Mounting the fork:

Quick release axle is released by pressing the push button between the "fork legs". Thereafter the forks are mounted in the bearing housing. Check that the axle is released by pulling down the fork.

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4. Installation of wheelchair brake:

Brakes are fitted using mounting clamp. The mounting clamp is fitted with the largest rounding around the wheelchair frame and the smallest on the brake. The brake is set so that brake push rod is 3-5 mm from the tyre when not applied. Tighten both screws.

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5. Fitting of armrests:

The armrest is mounted by inserting the hinged plastic adapter into the swivel element of the back tube. Note that the pin on the armrest must fit into the swivel element (photo 5.1). Then rotate the armrest so it is parallel to the seat and fold down so it snaps into the locking bracket at the front of the seat.

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6. Installation of leg supports:

The leg support is fitted by turning it 90 degrees out from the chair and pressing it down. It is then turned into place (photo 6.2). When the leg support is turned into place, it locks automatically. To remove the leg supports, push the black handle forward.

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6.1 Adjusting the height of the footplate:

The height of the footplate is adjusted by opening the locking clip, then adjusting the height and closing the clip again. Check that the tube of the footplate cannot move when the clip is closed. If this is the case tighten the unlocking screw on the back of the clip.

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6.2 Angle adjustment of the foot plate:

The angle of the footplate can be adjusted by loosening the screw on the outside of the footplate (photo 6.4). Then adjust the angle and retighten the screw.

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7. Mounting of push handles:

The push handles are mounted in the vertical back tubes. The clamping handle is mounted through the back tube and catches the thread in the sliding bushing of the push handle.

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8. Mounting the tip pedal:

If you wish to mount the tip pedals, you can do so on either the right or left side. The tip pedal is inserted into the horizontal tube at the bottom of the frame and screwed tight with the supplied bolt and nut. Antitip (not included) is attached in the same way.

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9. Adjustment of the seat depth:

Remove the yellow marked screws on both sides of the wheelchair, then the front part of the frame can be pulled out (photo 9.1). When you have found the desired depth, reattach the screws on both sides. The front support bushing for the armrest is moved by the same number of holes as the frame was extended by removing the bolt and nut (photo 9.2). The wheelchair brake is adjusted to the new seat depth (photo 9.3).

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10. Adjusting the wheelchair cushion:

The wheelchair cushion can be shortened to fit the selected seat depth. To do this, open the back cover of the cushion and remove the inner fabric cap. There are several cuts on the back of the cushion to mark the different seat depths. Using a hobby knife or similar, cut the cushion completely through to the desired depth. Then replace the inner fabric cap and cover.

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11. Adjusting the seat height:

If you want to adjust the seat height, move the mounting plate by loosening the four screws of the mounting plate and repositioning them (photo 11.1). The same height adjustment must be made on the wheelchair. This is done by moving the wheel position in the front fork or by replacing the front wheel with another size.

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12. Adjusting the center of gravity of the wheelchair:

On the Dolphin and Barracuda wheelchairs, the center of gravity can be adjusted, changing the weight distribution and handling characteristics of the wheelchair. To do this, remove the large nuts on the bushing and move the wheel bushing forward or backward. If you move the bushing forward, the weight on the front wheels decreases, making the chair easier to turn, but also increasing the risk of tipping backward. If you move the wheel bushing backwards, the weight on the front wheels increases. This setting is recommended for leg amputees.

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13. Adjusting the seat angle:

Adjust the seat angle by heightening the position of the mounting plate (as in step 11). Then adjust the angle of the bearing housing so that it's vertical again. To do this, loosen the two screws inside the frame (photo 13.2). Then adjust the round adjustment washer on the outside of the chair with an Allen key (photo 13.3). It is important that the bearing housing is completely vertical, as this has a great influence on the driving characteristics.

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Mobilex A/S